

Instructions for Handling Catheters / Stylets for Reprocessing



1. Wipe the entire device with a cloth and alcohol. Do not immerse the device in water or alcohol.
 - If sending in an RF Stylet, **REMOVE AND DISCARD SHARP**
2. Flush the lumen with a syringe full of alcohol. Then flush the lumen with a syringe full of air to remove as much liquid as possible. Do not screw the Luer cap on the Luer.
3. Package the cleaned devices into the original packaging. Ensure that the handle is secured and that the device does not get damaged.
4. Place the packaged device into a shipping box (sturdy outer package with adequate strength for shipment). Use either the original shipping box, the NES shipping box, or a similar shipping box with a sufficient depth to hold the devices.



5. Complete a Reprocessing Order Form and place it inside the shipping box (but not inside any blue or white product box), fill any open space with newspaper or other packing material and tape the box closed for shipment.



Northeast Scientific, Inc. will pay for return standard shipping on any box of catheters sent for reprocessing with a total order of >\$1,000. Batch your catheters for shipment accordingly.

REPROCESSING ORDER FORM **nescientific**
smart healthcare™

To be used when shipping devices to Northeast Scientific, Inc. for reprocessing.
Complete this form and place it inside the shipping box. Please complete a new form for each shipping box.
CONTENTS: Used Medical Devices. Property of the sender.
Ship to: Northeast Scientific, Inc. 2142 Thomaston Ave. Waterbury, CT 06704

Customer Name: Your Business Name Account Number: Your Account Number

Date Shipped: 01 / 01 / 2018

Quantities Shipped:

ClosureFast CF7-7-60	<u>5</u>	RF Stylet: <u>2</u>	TOTAL DEVICES IN BOX: <u>16</u>
ClosureFast CF7-7-100:	<u>6</u>		
ClosureFast CF7-9-60:	<u>3</u>		

Return Shipping Address: Your Business Address Street and Suite Number Town / City State Zip Code


By shipping the enclosed devices for reprocessing, the account irrevocably commits to pay for each device successfully reprocessed and return shipped pursuant to the terms of the Reprocessing Services Agreement with Northeast Scientific Inc.

Return shipping requested: Standard 3-day* Expedited 2-day Expedited Overnight

*If the total reprocessing charge for this order is > \$1,000, standard 3-day shipping will be at no charge. If the total charge is < \$1,000, or if expedited shipping is requested, standard shipping charges will apply.

F-1148 Rev. 1 Reprocessing Order Form ©2018 Northeast Scientific, Inc. All Rights Reserved

6. Affix the pre-addressed, prepaid shipping label to any corner on the shipping box top and call FedEx at 888-777-6040 to pick up your shipment.

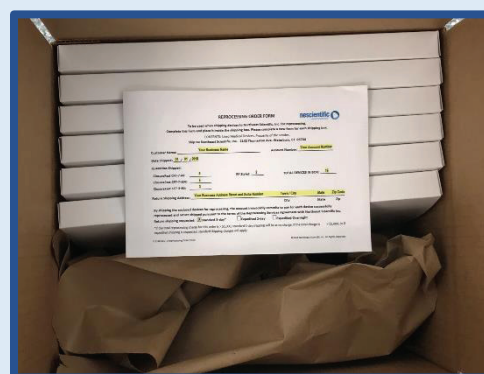

QUESTIONS? For questions concerning your order, or for additional quantities of shipping labels or reprocessing order forms, please contact:

Northeast Scientific, Inc.
2142 Thomaston Avenue
Waterbury, CT 06704
Phone: 203-756-2111
Fax: 203-757-5532
info@smarthealth-care.com
sales@smarthealth-care.com
https://www.smarthealth-care.com



The NES Reprocessed Varicose Vein RF Catheter is intended for endovascular coagulation of blood vessels in patients with superficial vein reflux. The NES Reprocessed Varicose Vein RF Stylet is intended for use in vessel and tissue coagulation including: Treatment of incompetent (i.e., refluxing) perforator and tributary veins. Please see the Instructions for Use for a complete listing of the indications, contraindications, warnings and precautions.

CAUTION: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

ClosureFast® and ClosureRFS® is a registered trademark of Covidien LP Limited Partnership.

Reprocessing of ClosureFast® catheters and ClosureRFS® stylets is performed by NES and is not affiliated in any way with the original manufacturer.

©2018 Northeast Scientific, Inc All rights reserved.
This literature is intended for distribution only in the U.S.