

# INSTRUCTIONS FOR RETURNING TURBO ELITE CATHETERS FOR REPROCESSING



**STEP 1**  
Wipe the entire shaft of the device with a cloth and **70% Alcohol**.




**STEP 2**  
Flush the lumen with a syringe full of **70% Alcohol**.


Flush the lumen with a syringe full of air to remove as much liquid as possible.



**STEP 3**  
Feed the device back into the original coil, pushing only small lengths at a time to avoid kinking.

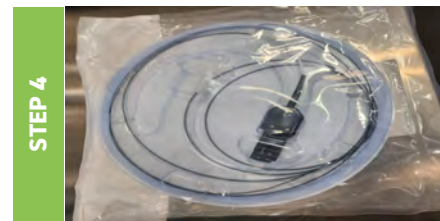

**STEP 4**  
Place the coiled device in its original packaging or a plastic bag.

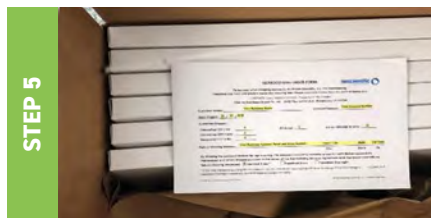

**STEP 5**  
Place the device into a sturdy shipping box, filling any void with paper or other soft material.

Print and fill out a Reprocessing Return Form and place inside the shipping box on top of the catheters.



**STEP 6**  
Affix the prepaid FedEx shipping label to any corner on the shipping box top and call FedEx at 888-777-6040 to pick up your shipment.

No biohazard markings are to be placed on the inner or outer packagings.

Reference Exceptions 49 CFR 173.134(b)(1)(4) and 49 CFR 173.134(b)(1)(5)

**If returning RF or IVUS catheters, please follow the RF or IVUS specific instructions.**